



THE HONG KONG STANDARDS AND TESTING CENTRE LTD.  
**CERTIFICATE OF COMPLIANCE**

**Certificate Number:** EMC0805091

**APPLICANT:**

Taising Hearing Amplifier Ltd  
FLAT C, 15/F, PHASE 1  
GOLDEN DRAGON IND CTR  
152-160 TAI LIN PAI RD. KWAI CHUNG N.T. H.K.

**DESCRIPTION OF SAMPLE:**

Product: Rechargeable ear hearing amplifier  
Manufacturer: Taising Hearing Amplifier Ltd  
Model No.: TS-8328  
Brand Name: N/A  
Origin: N/A

**Applicable Standard(s) with amendments:**

EN61000-6-3: 2007  
EN61000-6-1: 2001  
EN55013: 2001 +A1: 2003 +A2: 2006  
EN55020: 2007  
EN61000-3-2: 2006  
EN61000-3-3: 1995 +A1: 2001 +A2: 2005

REFERENCE TEST REPORT NUMBER: HM162579

*This is to certify that the submitted sample has been tested in accordance with and found to be in compliance with the said investigation.*

Date: 2008-10-14



Dr. LEE Kam Chuen

ElectroMagnetic Compatibility Department  
For and on behalf of  
The Hong Kong Standards and Testing Centre Ltd.

# rechargeable ear **ear** TS-8328

## hearing ((( amplifier

### OPERATING INSTRUCTIONS

#### How to charged up Powerful Cube & Rechargeable Ear Amplifier

##### > A: Powerful Cube(CUBE)

- "Cube" input is DC5.0V  $\geq$  200mA USB connection;
- Content Li-Polymer battery;
- Use USB cable to connect computer or USB adapter.

Insert USB plug in "Cube";  
When start charged red light will be "ON"  
It's charged is process;  
When full charged Red light will be "OFF";  
Best charged time is 3-5 Hours.  
When fully charged it can be supply power source for "EAR" around 4 times (each time is charged 2 Hours).

"Cube" is need weekly charged ONCE time to keep battery fresh.

##### > B: Rechargeable Ear Amplifier (EAR)

To open "Cube" door "PUSH" and place "EAR" into cavity;  
Be sure "EAR" is turn OFF "O" position;  
Close door, "Cube" begin to charged for "EAR" and Green light will be "ON";  
When fully charged Green light will be "OFF";  
Best of charged time is up to 2.5 Hours;  
When complete charged "EAR" can be use 8-24 Hours.

##### > C: "Cube" & "Ear"

Both can charged together;  
Charged is same as above (A) & (B) step.  
First to use need to process (C) step.  
Before process charged, take out "EAR" protect seal.



Powerful Cube



USB cable



USB AC adapter



USB adapter



USB plug in



Rechargeable Ear Amplifier (EAR)



#### CAUTION

1. The products is not water proof therefore the users must take care to prevent water form entering the unit. Should water enter the Incredible Ear Amplifier remove the battery and place the unit in a warm place to dry.
2. It is important to increase the volume control very gradually to avoid sudden increase in sound.
3. This product is not a medical device.